



Independence Made Easy



Independence Made Easy



Why should you let getting up and down the stairs stop you from having a fully independent lifestyle? With a tailor-made Brooks Stairlift, you can stay in your home, and continue to enjoy your independence.

In 1972, Mr. Frederick Brooks, a former engineer, designed a stairlift for his wife who suffered from rheumatoid arthritis, and subsequently found using the stairs difficult.

This 'original' stairlift attracted much attention due to its design and high standards of engineering, and over forty years later Brooks Stairlifts are still following these high standards set by Frederick Brooks. Combining today's technology with a strong tradition of reliability and comfort, each and every Brooks Stairlift is guaranteed to help you make the most of your home.

Brooks Stairlifts are designed to suit your individual home and lifestyle, and are engineered with a range of features that make them simple to use. Through innovation and investment, Brooks have always delivered products with the customer in mind, aiming to provide the perfect solution for your needs.

BROOKS
STAIRLIFTS

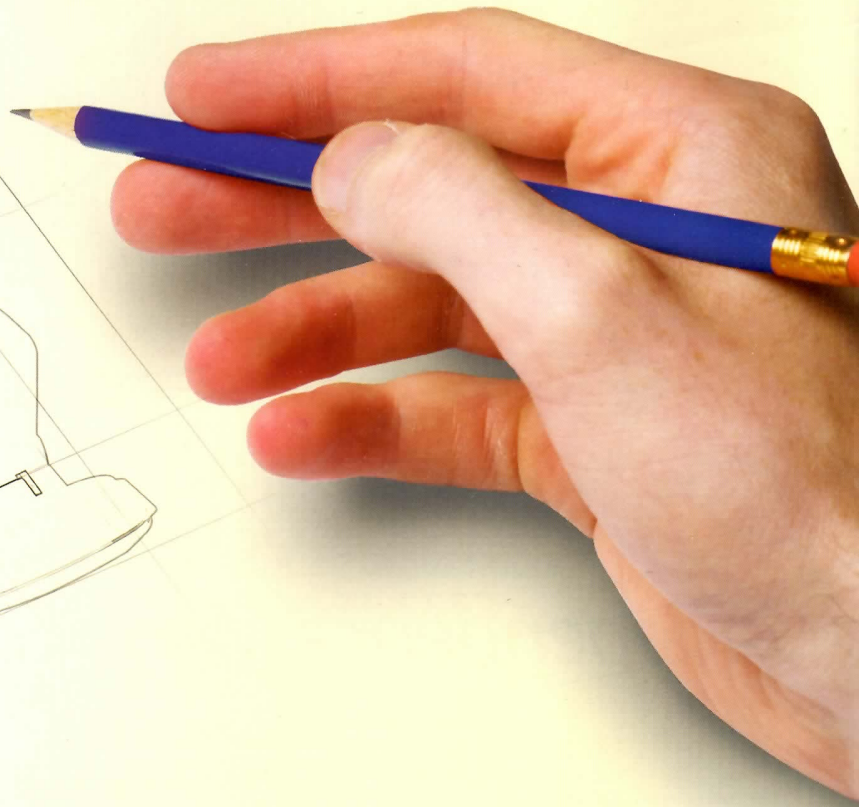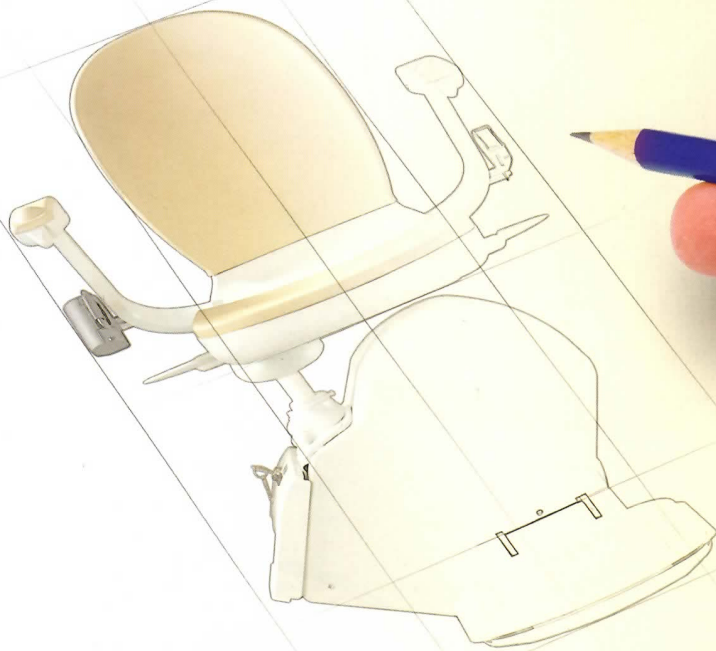



Brooks Stairlifts blend seamlessly into your home, and the smooth start and stop mechanism allows you to glide effortlessly up and down the stairs.

Brooks Stairlifts are designed with simple and effortless operation, ease of use and comfort in mind.



Brooks Stairlifts are installed and serviced by local authorized dealers, who have been highly trained to give you peace of mind.



Attention to detail during installation ensures that your stairlift is fitted precisely to your staircase, and takes up the minimum amount of space.

Professionalism and Care

From the moment your Brooks Stairlift is installed, we want to ensure that you are happy with it and confident in its use.

Your local, authorized Brooks Dealer will first carefully assess your needs and ensure that your chosen stairlift solution is both suitable and affordable.

Brooks Stairlifts are precision manufactured, designed and assembled, conforming to the highest Safety Standards.

After thoroughly checking that everything works just as it should, all the features will be fully explained and demonstrated to you to ensure that you are happy and confident in using the stairlift.

We are sure that you will enjoy peace of mind in knowing that you have been looked after every step of the way.

*"From start to finish the service was first class.
I no longer have to think about going upstairs, I just do it."*

Mrs. White, Columbus, Ohio



Folding track available for situations where track protrudes into walkway at the bottom of the stairs (usually a doorway)



A retractable safety belt is fitted to all Brooks Stairlifts so you feel secure at all times.



Safety sensors on the footrest and carriage stop the lift automatically and very gently should there be any obstruction on your staircase.



Directional paddle switches (one in each arm rest) give you complete control. Designed to operate with ease, should you have limited dexterity.



A lockable swivel seat means you can get on and off your stairlift without twisting your body. Once turned it also provides a stable base and safety barrier at the top of your stairs.




A simple digital display allows you to see the exact status of the stairlift at all times.



Two remote controls are provided so that you have the convenience of being able to call or send the lift up or down the staircase.



A lockable on/off switch provides additional "child-friendly" safety

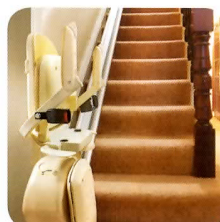

Slim fold away design has folding arms, seat and footrest so other people in your home can continue to use the staircase as normal.



All electrical systems use fully sealed cables, and all plastic materials are flame retardant and UV resistant.




Optional Folding Track

In situations where there may be an obstruction at the foot of the staircase (usually a door), or where the stairlift could cause a tripping hazard, a folded rail is required to 'fold back' away from the obstruction when the stairlift is not in use. (Not available for commercial or outdoor use.)

Anatomy of the Brooks Indoor and Outdoor Stairlifts

What makes the Brooks the leading stairlift in the world? It is a combination of thoughtful design, top class engineering, reliability and user-friendly features.


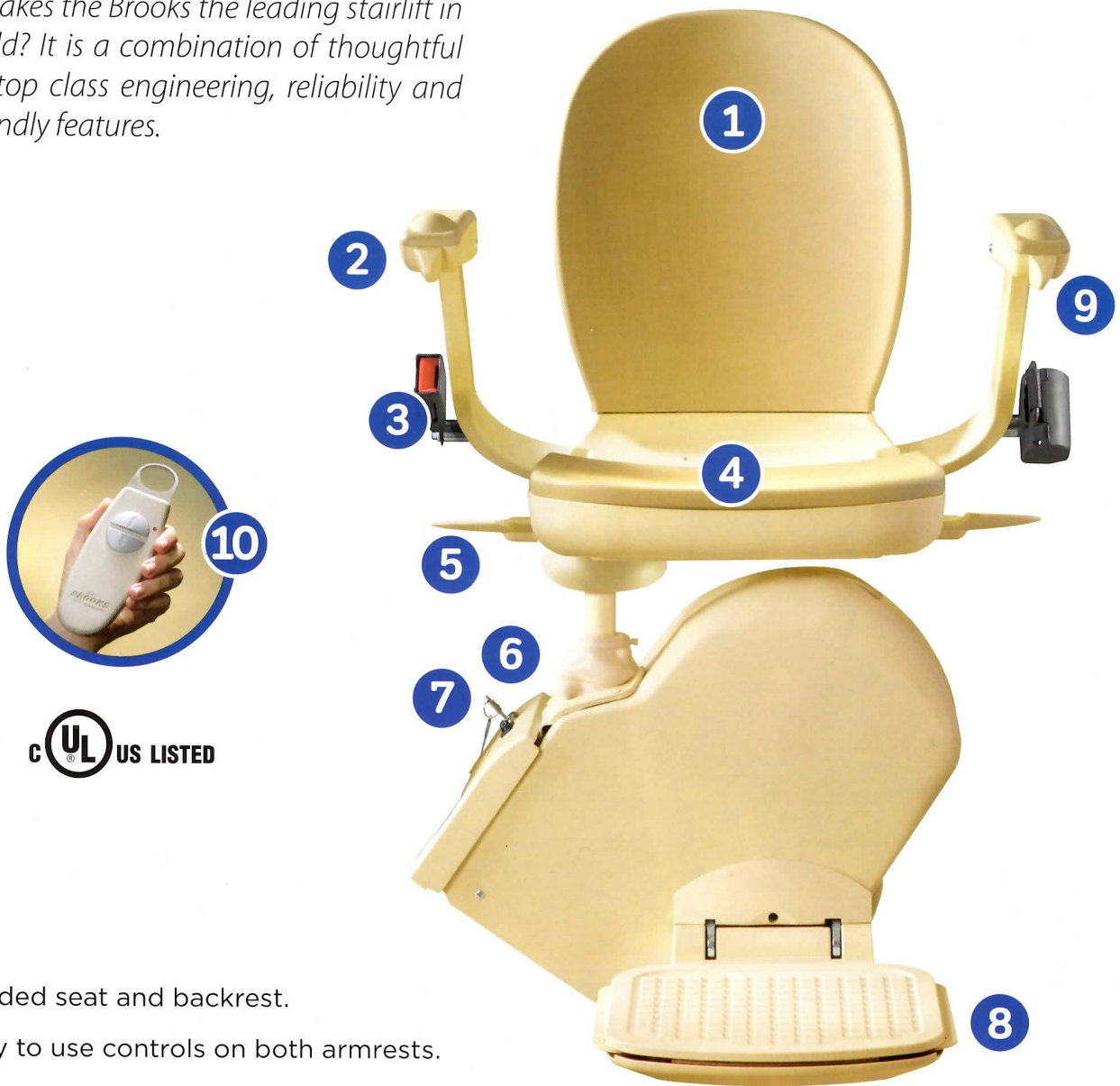


- 1 Padded seat and backrest.
- 2 Easy to use controls on both armrests.
- 3 Secure, retractable seat belt.
- 4 Folding arms, footrest and seat allow easy access to the stairway.
- 5 Swivel seat - no need for twisting the body to get on and off the lift.
- 6 Lockable on/off switch allows the user to prevent others (e.g. children) from using the lift.
- 7 Diagnostic digital display informs the user of the status of the lift.
- 8 Five safety sensors on the footrest and carriage stop the lift automatically if any obstruction is encountered.
- 9 Safety switch beneath down-side armrest, prevents operation when in 'up' position.
- 10 Hand-held remote controls allow users to 'call' or 'send' the lift up and down the staircase.



All Brooks Stairlifts
are designed to
fold up giving other
people safe access
to the staircase
when the lift is not
in use.

Brooks Straight Indoor and Outdoor Specifications

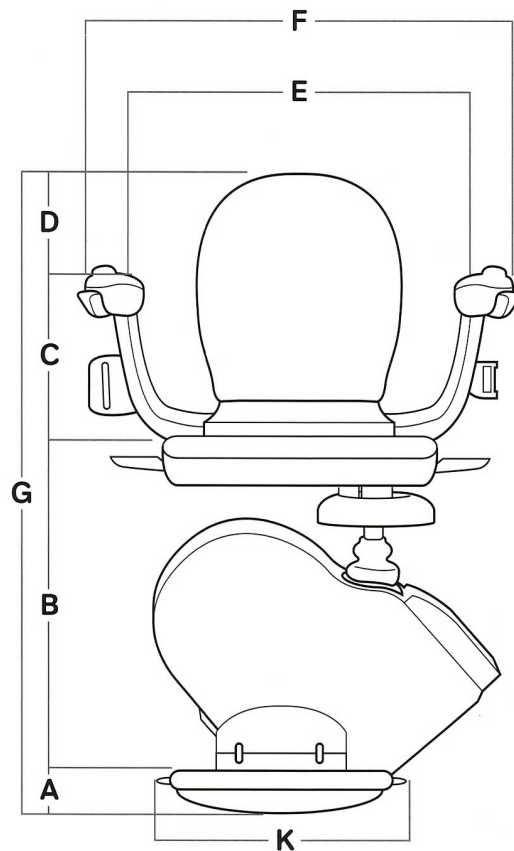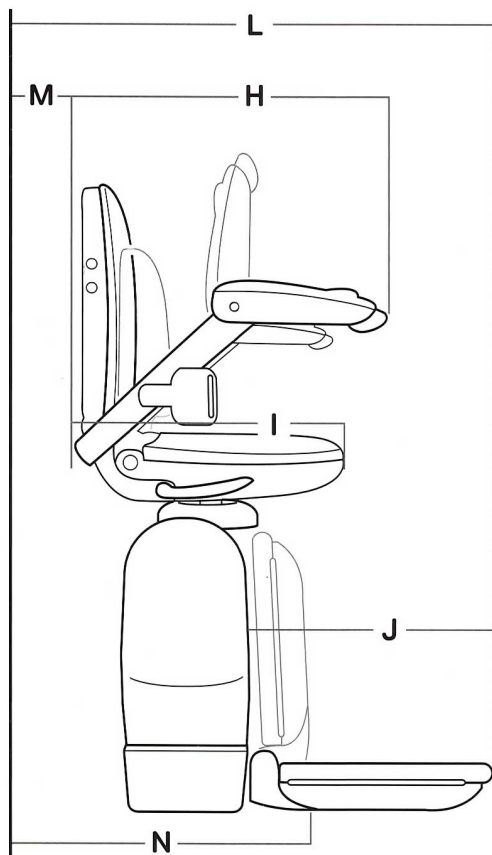

Dimensions

	mm	inch
A Floor to top of footrest	95	3¾
B Top of footrest to top of seat	445	17½
C Top of seat to top of arms	250	9¾
D Top of arms to top of seat back	190	7½
E Width between armrests (wide arm version)	445 (495)	17½ (19½)
F Overall width	605	23¾
G Overall height	980	38½
H Length of arms from seat back	470	18½
I Length of seat base from seat back	435	17
J Length of footrest	325	12¾
K Width of footrest	310	12¼
L Front of footrest to stringer	570	22½
M Back of seat to stringer	13	½
N Folded depth	310	12¼

Specifications

Motor Output Speed

No greater than 0.15m/s
or 29ft/min

Method of Drive

Rack and Pinion

Power Supply

24V DC (battery)

Maximum Capacity

300 lbs. (Optional 350 lbs.)

Track Extruded

Aluminum

Mains Supply

120 - 240V AC

Measurements are only a guide and subject to site and client survey.



The Swivel Seat means that there is no need to twist awkwardly when rising from the stairlift at the top of the stairs and the seat also acts as a safety barrier.

Make the most of your home with a BROOKS Outdoor Stairlift

Brooks Stairlifts have slimline motors that provide a soft-start and soft-stop action. The slimline motor provides years of smooth, steady service, while its host of innovative features offer exceptional safety and comfort.



The Brooks Outdoor Stairlift is equipped to withstand most weather elements. Below are a few of the features that make the outdoor stairlift weather-resistant:

- ✓ **The seat post** is made of 100% stainless steel
- ✓ **Rack and pinion drive** is Nitrotech coated
- ✓ **All electrical systems** use fully sealed cables
- ✓ **All plastic materials** are flame retardant and UV resistant





*"We love the freedom and independence
our Brooks Stairlift gives us"*

Mr. and Mrs. Burrows, Dallas, Texas



Brooks Stairlifts Ltd.

6450 Kingspointe Pkwy, Unit 1
Orlando, FL 32819

All materials copyright © 2016 Brooks Stairlifts Ltd. All rights reserved.

No part of this material may be used for any purpose without prior written permission. Use without permission is expressly prohibited by law, and may result in civil and criminal penalties. Photographs and diagrams are used for illustrative purposes only. All information including dimensions, specifications and prices are correct at the time of printing and are subject to change.

For more information or to arrange a visit,
please contact:



StairLift®
HEADQUARTERS

RAMP®
HEADQUARTERS

1-866-549-8893